kinetec®

# THOMAS SPLINT *ATTELLE DE THOMAS*

ORTHOPAEDIC TRACTION  
TRACTION ORTHOPÉDIQUE

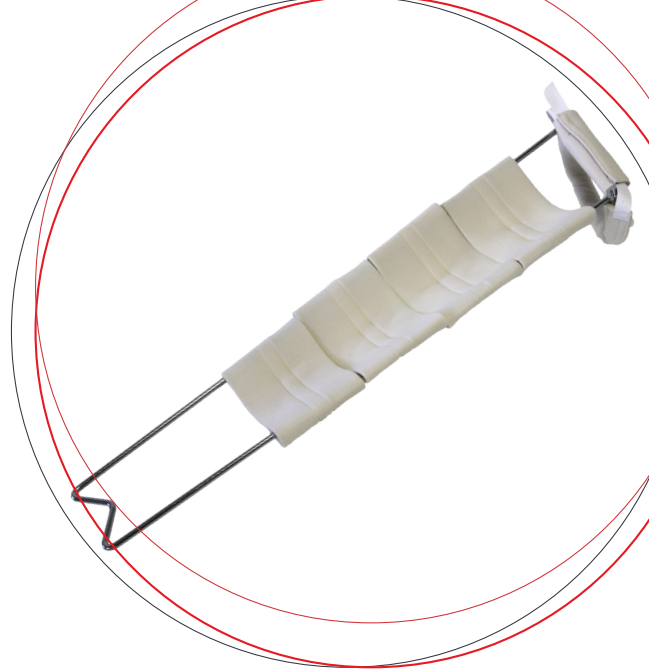
# ORTHOPAEDIC TRACTION TRACTION ORTHOPÉDIQUE

The Kinetec Thomas Splints are modular, allowing you to select the appropriate leg support length and thigh hoop width per patient. The thigh hoops connect to the leg supports by our innovative magnet system ensuring that the Thomas Splint is quick and easy to assemble without tools.

- Thomas splints can be used for either right or left legs.
- They are made from chrome-plated steel for easy cleaning and durability.

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## LEG SUPPORT AND THIGH HOOP ÉLÉMENT CRURAL ET ÉLÉMENT JAMBIER

Kinetec Thomas Splints are available in 4 sizes of thigh hoop and 4 sizes of leg support. Each thigh hoop will fit on to each leg support so you can choose the most appropriate combination (except Size 1 Thigh Hoop should not be used with Size 4 Leg Support and Size 4 Thigh Hoop should not be used with Size 1 Leg Support).

Use the patient's uninjured leg for taking measurements.

To select the the Leg Support, measure inside leg from groin to the bottom of the heel. Add another 20cm to allow for ankle plantar flexion in the splint and for the application of a windlass.

To select the Thigh Support, measure the thigh circumference near the groin at an oblique angle.

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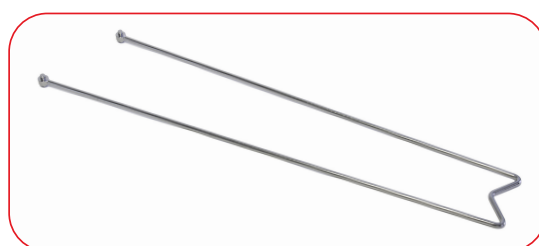
\* Including 20 cm allowance - Comprenant une marge de 20 cm.



## ORTHOPAEDIC TRACTION TRACTION ORTHOPÉDIQUE

### THIGH HOOP FOAM COVERS MOUSSE CRURALE

To provide additional comfort to the patient, the Thigh Hoops can have a foam tube padding added. These are supplied in one length in packs of 10 and are long enough to cover the largest Hoop Cover.

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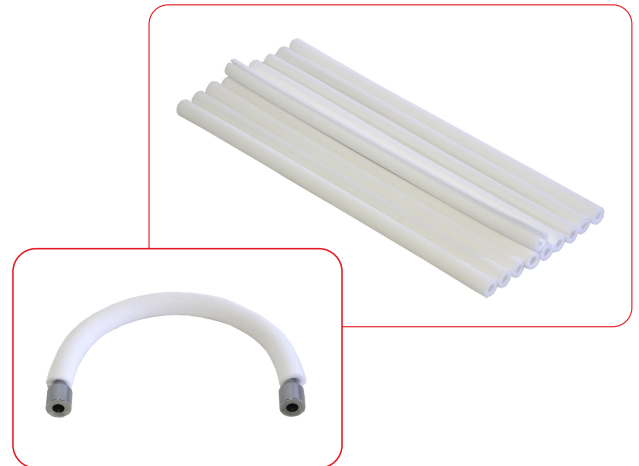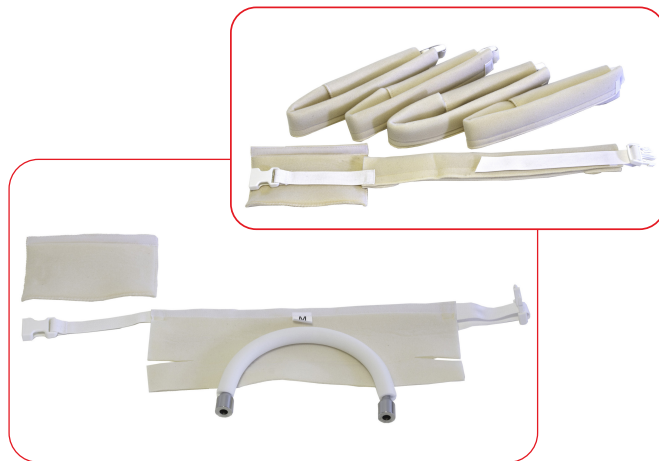### HOOP COVERS PROTECTION CRURALE

The Thigh Hoop Cover sizes, correspond with the Thigh Hoop sizes as follows and comes as a pack of 5.

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### SLING MANCHONS

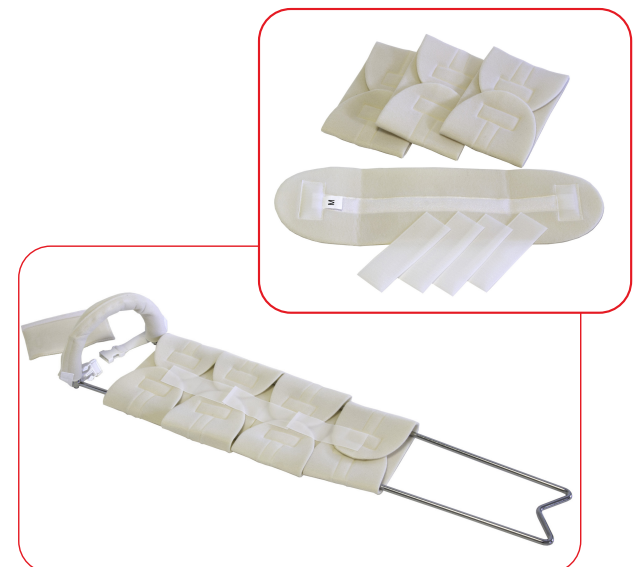
Made of foam and with hook and loop fixing, these single patient use Slings can be used in different size combinations to create a soft, comfortable support for the injured leg.

Hook strips are provided to keep the individual slings from slipping apart. (The Kinetec Thomas Splint Slings can also be used on other Thomas Splints.)

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# ORTHOPAEDIC TRACTION TRACTION ORTHOPÉDIQUE

## WINDLASS CLÉ DE SERRAGE

To provide the clinicians with the capability to tighten the cord at the end of the traction, a pack of 5 windlasses is available. These steel windlasses are strong and can be easily cleaned.

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## SUSPENSION RINGS ANNEAUX DE SUSPENSION

If the clinician wishes to apply suspension to a patient's leg whilst they are wearing the Thomas Splint, the Suspension Rings will provide a solution.

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MedFac UK Ltd
Kinetec SAS