EN - INSTRUCTIONS FOR USE

Intended use

Manosplint® Cushion Foam is a foam used in the design of removable splints.

Warnings

Manosplint® Cushionfoam should only be used for the design of removable immobilization splints.

The splint must be removable by the patient to allow them to air the skin daily, especially in case of sweating.

The immobilized limb must be dry before the splint is applied and at all times thereafter.

Do not attempt to clean or wet the Cushion Foam, use a new one.

Kinetec recommends using Manosplint® Cushion Foam in combination with products from the Manosplint® and Runlite ranges.

Any serious incident related to the device should be reported to the manufacturer by mail at contact@kinetec.fr and the competent authority of the Member State in which the user and/or patient is located.

Symbols used

	Follow the instructions for use		CE marking of conformity		Medical device		Manufacturer		Manufacture batch number		Device reference		Temperature limits during storage and transport
---	---------------------------------	---	--------------------------	---	----------------	---	--------------	---	--------------------------	---	------------------	---	---

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Contraindications

Manosplint® Cushion foam should not be used to create a closed circular immobilization or any type of enclosed environment for the skin.

Do not apply cream, ointment, or perfume to the immobilized limb.

Side effects

In rare cases, certain adverse effects can appear, such as:

- allergic skin reactions
- skin redness
- itching

Intended use environment

Manosplint® Cushion Foam is designed for use in a dry environment.

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